

# GRAFT APPLICATION: Package to Patient in 3 Easy Steps




## EPIEFFECT™

EPIEFFECT™ is a lyophilized human placental-based allograft membrane that includes amnion, intermediate, and chorion layer. Following an adequate wound bed preparation, carefully follow the three simple steps below using aseptic technique:

### STEP ONE OPEN

- Open the peelable corner of the non-sterile outer pouch and present the inner pouch onto the sterile field.
- Slowly peel a corner of the inner peel pouch and grasp EPIEFFECT with fingers or non-toothed, sterile forceps.

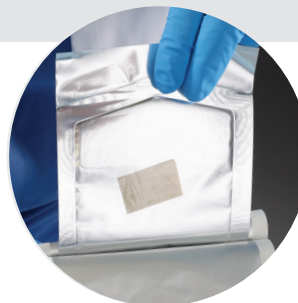

### STEP TWO PREPARE

- Identify the orientation. EPIEFFECT has a pattern on each side: The larger pattern should be placed down on the wound, and the smaller pattern should face up.
- If needed, trim to the appropriate size to cover the wound site using sterile scissors.



Larger pattern side faces  
DOWN



Smaller pattern side faces  
UP

### STEP THREE APPLY

- Place EPIEFFECT on the wound site or soft tissue deficit and hydrate with sterile saline, if desired. As the tissue hydrates, the pattern will fade.
- Once placed, the tissue can easily be repositioned.
- EPIEFFECT is compatible with offloading, compression, Negative Pressure Wound Therapy, and Hyperbaric Oxygen Therapy.



Tissue pictures are representative. Pattern may vary slightly.

Cover with appropriate wound dressings. For more details, please refer to the Instructions For Use.



Scan here to view the EPIEFFECT  
application video.  
<http://bit.ly/EPIEFFECTapply>